LIFAair®

LIFAair LM991 KN95-hengityssuojain

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Lifa Air Oy Ltd.

Osoite: Vellamonkatu 30B, 00550 Helsinki

Puh: +358 9 394 858

E-mail: myynti@lifa.net

www.lifa.net

Valmistaja

Dongguan LIFAair Technology Ltd.

Osoite: No. 2, East Industry Road, Songshan Lake Science & Technology Industrial Park,
Dongguan, 523808, China

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Tillverkare

Dongguan LIFAir Technology Ltd.

Adress: No. 2, East Industry Road, Songshan Lake Science & Technology Industrial Park,
Dongguan, 523808, China

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Manufacturer

Dongguan LIFAir Technology Ltd.

Address: No. 2, East Industry Road, Songshan Lake Science & Technology Industrial Park,

Dongguan, 523808, China

www.lifa-air.com

Use

LIFAir LM991 Face Mask is intended for professional use to protect against the coronavirus.

Description of the Face Mask

Product Name:	LM991 Face Mask
Protection level:	KN95
Color:	White
Exhalation Valve:	No
Adjustable nose clip:	No
Strap attachment:	Plastic connector
Straps adjustable:	Yes

Instructions

	Remove the face mask from its package and open it with clean hands.
	Place the face mask on your face.
	Pull the straps behind your head while holding the mask on your face with your other hand. Place the upper strap above your ears and lower strap below your ears. Ensure that the straps are not twisted.
	If necessary, tighten by pulling the strap between plastic part. You may hold the plastic part with your other hand for easier adjusting. <i>(If the strap detaches, you may slip it back to its track)</i>
	Ensure that the face mask fits and is tight. If it is not, you may re-adjust the straps placement and tightness again. Ensure the tightness after re-adjustments. Please only touch the mask with clean hands.

Notice! The face mask is **not reusable**, and it must be disposed of appropriately after use. Hands must be washed thoroughly always before removing the face mask and after disposing of the face mask.

Notice! This product must not be used near open fire.

Notice! Facial hair may prevent proper fit of the face mask. In case the mask does not fit tightly, must user trim or remove facial hair.

Storage

Store in a cool, dry place protected from direct sunlight. Remove face mask from its airtight packaging only before placing it on your face.